




- Biocompatible-made of all natural Chitosan-based polymers
- Minimizes bleeding and edema after surgery
- Easily conforms to nasal anatomy
- Offerseasier placement with a superior field of view
- No special storage conditions required

### PRODUCT DESCRIPTION:

**PosiSep® X** is a hemostatic intranasal splint designed to minimize bleeding and edema and to prevent adhesions between the septum and the nasal cavity. **PosiSep® X** is placed in the nasal cavity after surgery or trauma. The **PosiSep®** line is constructed of a patient-comfortable sponge which is manufactured from naturally occurring Chitosan polymers. Subsequent irrigation will cause the product to fragment, thus optimizing patient tolerance and comfort.

- Fragments by way of gentle irrigation and aspiration
- Easy to apply-no mixing or additional prep time necessary

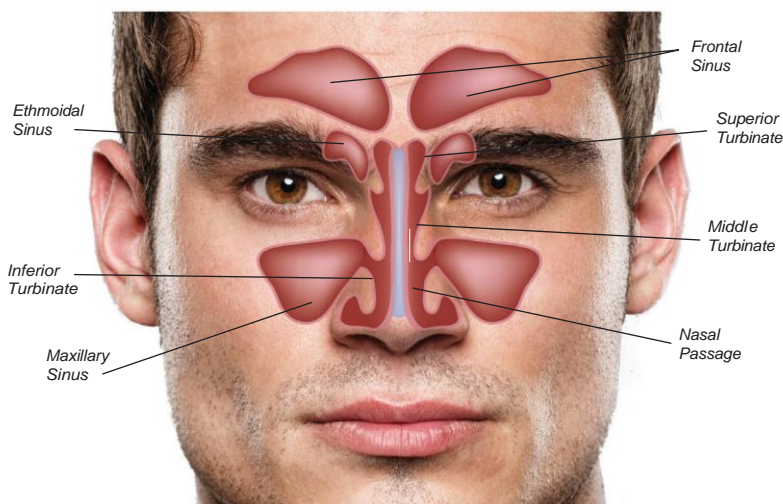

\*diagram not to scale\*

**PosiSep® X** Hemostat Dressing/Intranasal Splint is indicated for use in patients undergoing nasal/sinus surgery as a space occupying splint and hemostat to:

- Separate tissue or structures compromised by surgical trauma;
- Separate and prevent adhesions between mucosal surfaces during mesothelial cell regeneration in the nasal cavity;
- Help control minimal bleeding following surgery or trauma;
- Help control minimal bleeding following surgery or nasal trauma by tamponade effect, blood absorption and platelet aggregation; and
- Act as an adjunct to aid in the natural healing process.

PosiSep is indicated for use as a nasal hemostat to treat epistaxis.

PosiSep is intended for use under the direction of a licensed healthcare provider.



### Before Hydration



### After Hydration



When PosiSep X is hydrated, it expands to fill the space preventing post-op adhesions.

### What is Chitosan?

Chitosan (ky-toe-san) comes from a substance known as chitin, an amino-polysaccharide removed from the powdered shells of crustaceans.



Postboks 456 5501 HAUGESUND  
Tlf. 5270 4140 Fax 5270 4144

[mail@norsk-medikal.no](mailto:mail@norsk-medikal.no) [www.norsk-medikal.no](http://www.norsk-medikal.no)

### ORDERING INFORMATION

PosiSep® X is available in two sizes. See ordering information below.

Part #:	Description:
9215674	PosiSep® X Hemostat Dressing/Intranasal Splint 1.5 cm x 5.1 cm/ 8pk
9215664	PosiSep® X Hemostat Dressing/Intranasal Splint 1.3 cm x 3.8 cm/ 8pk

### How Supplied:

PosiSep® X is packaged eight (8) sterile, single use units per carton. One (1) sterile unit contains one (1) Hemostat Dressing/Intranasal Splint. PosiSep® is terminally sterilized by gamma irradiation.



manufactured by:

5000 Township Parkway  
St. Paul, MN 55110  
Tel 866-612-2568  
[global.hemostasisllc.com](http://global.hemostasisllc.com)

PosiSep® is a registered trademark of Hemostasis, LLC  
© 2015 Hemostasis, LLC  
PS-0016 Rev B (EU) 06.02.15